

Mobile 12-Lead ECG Quick Reference Guide

The easiest way to acquire a 12-lead ECG, making critical data accessible in more places, more often.

Getting Started

- 1 For first time setup only:** Set up a KardiaPro account and invite users. **You will need the unique 12-character activation code printed below. Enter the code at kardiapro.com/activate.**
- Use the QR code below to download the KardiaStation app on a compatible smart device. Full list of compatible devices can be found at alivecor.com/compatibility/pro.

Use the QR code to download the KardiaStation app before use



Included components


- Kardia 12L ECG Module with (2) AAA Alkaline Batteries
- Patient Lead Wire
- Snap-to-Tab Adapters (10)
- Screwdriver for battery replacement
- 100 electrodes

Enter code to create KardiaPro account

Visit alivecor.com/Kardia12L for complete Instructions for Use, and all Warnings and Precautions.

KARDIA™ 12L

This set of lead wires meets the requirements of the standard, ECG trunk cables and patient lead wires (ANSI/AAMI EC53). This set of lead wires is intended for multi-patient use.

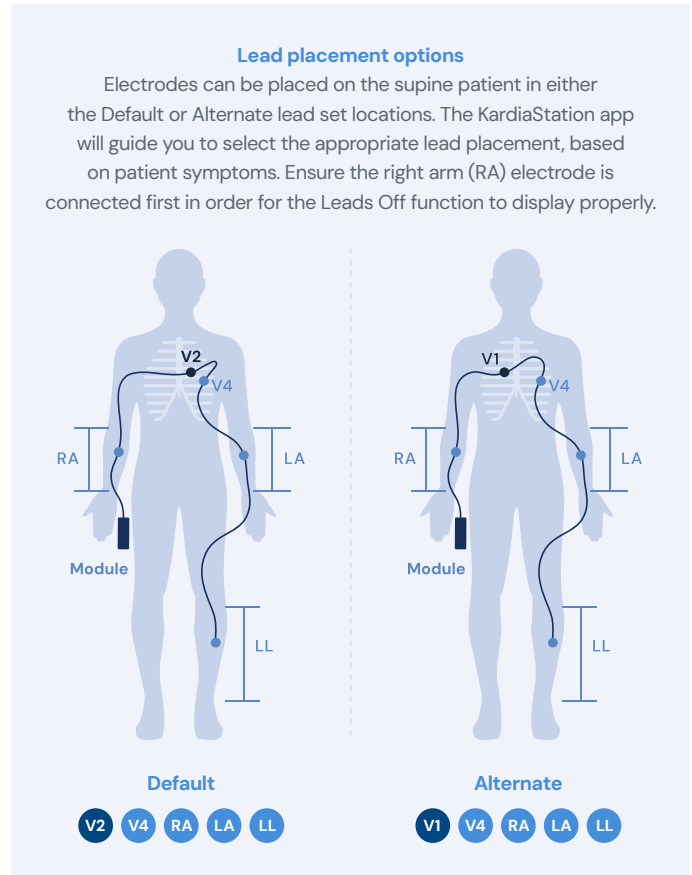

For additional information, or to request a paper copy of the Instructions for Use, contact: clinicalsupport@alivecor.com or call +1 (844) 620-1060.



alivecor.com/quickstart

27LB09.02